
Safety Chrom

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Safety Chrom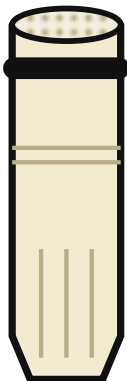

DE	Gebrauchsanleitung	S. 5
EN	Instructions for use	p. 8
FR	Mode d'emploi	p. 11
IT	Istruzioni per l'uso	p. 14
ES	Instrucciones de uso	pág. 17

1071

0,2 µm


Porenweite · pore size

log 7

99,99999 %

6 M


max. 4 000 l

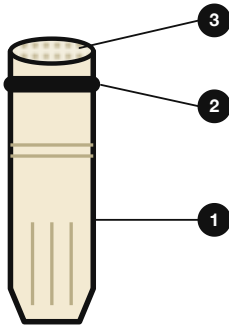5 min

4 000 l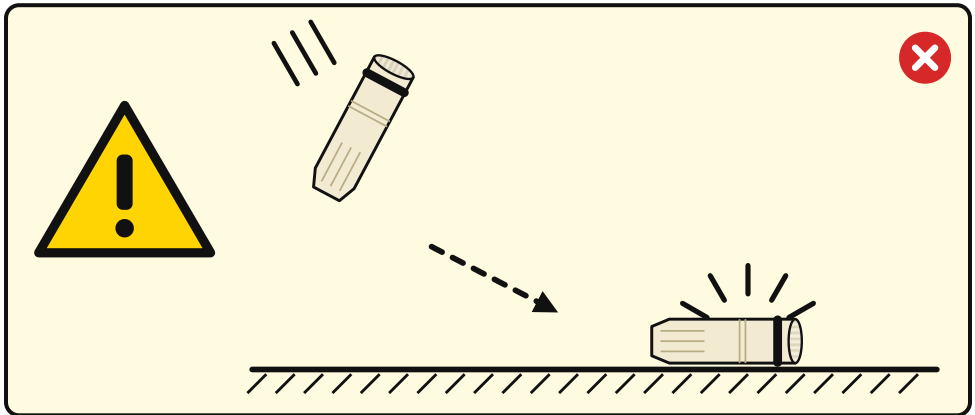

> 60 °C







< 5 °C




max. 4 000 l

B Filter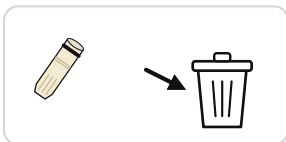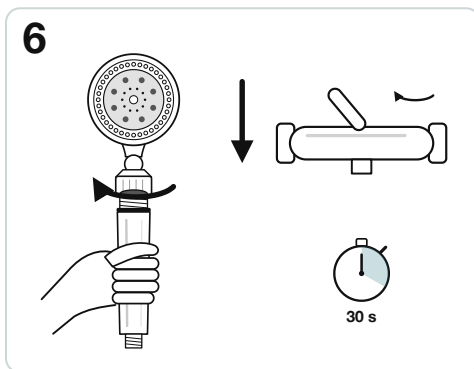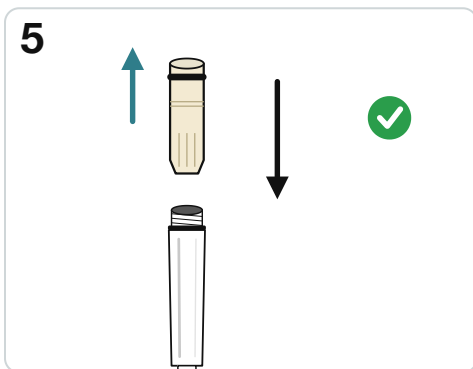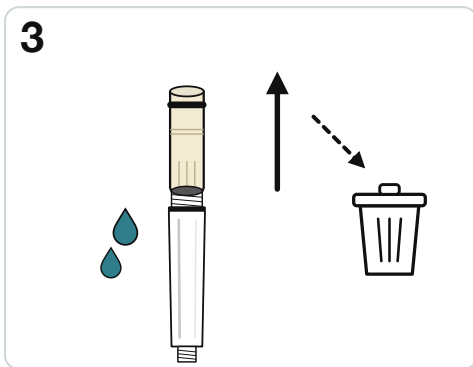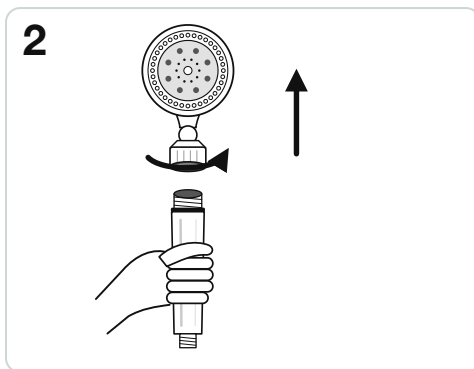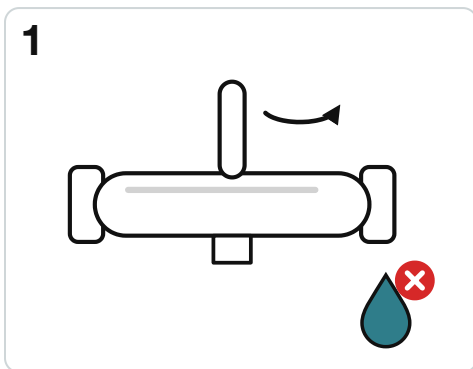

1071
Austauschfilter Safety Chrom ·
Replacement filter · Filtre de
rechange · Filtro di ricambio ·
Filtro de recambio

Safety Chrom

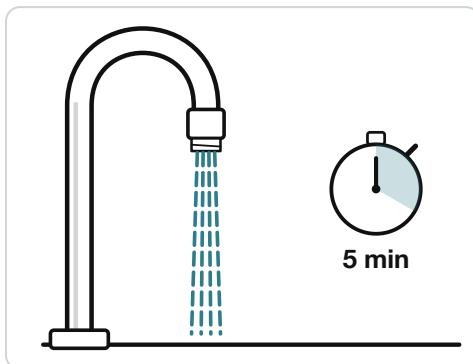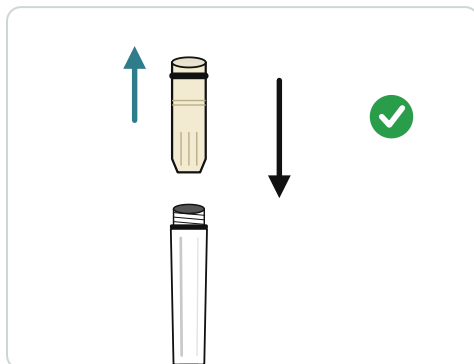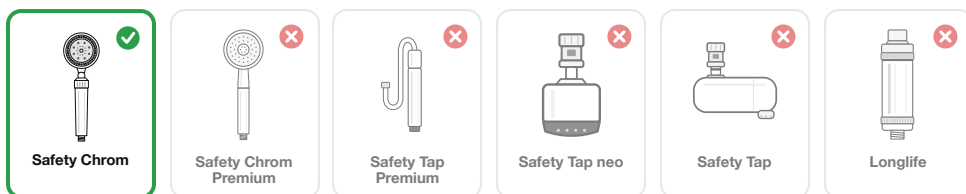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INTENDED USE

The replacement filter is a sterile filter cartridge (hollow-fibre membrane, 0.2 µm) for the Safety Chrom anti-*Legionella* shower head. It retains *Legionella*, *Pseudomonas* and other bacteria from the shower water; every cartridge is individually integrity-tested. Only the cartridge in the handle tube is replaced – the shower head is kept.

- 1 Filter cartridge
- 2 O-ring
- 3 Membrane side – facing up, do not touch

Fits: Safety Chrom

Does not fit: Safety Chrom Premium, Safety Tap Premium, Safety Tap neo, Safety Tap, Longlife

SAFETY INSTRUCTIONS

Do not drop! Impacts and falls can cause fine, invisible cracks in the housing – especially at the transition between shower head and handle – or in the filter membrane. Filter performance and tightness can then no longer be guaranteed. Do not continue to use the product after a fall.

- **Temperature:** Up to 60 °C continuously, up to 70 °C briefly (max. 30 minutes). Unscrew the shower head before thermal disinfection or chlorination of the pipes. Afterwards, let the water run at a washbasin for approx. 5 minutes.
- **Hygiene:** Do not touch the membrane side of the cartridge. Only hold a new cartridge by its light-coloured housing.
- **Replacement interval:** Replace the filter cartridge after 6 months or 4,000 litres at the latest, whichever comes first – even if the water still flows well.
- **Installation:** Hand-tighten only. Do not use tools or sealing tape.
- **Do not tamper:** Do not open, backflush or reuse the cartridge. Do not use damaged or leaking products.
- **Storage:** Store frost-free, dry and in the original packaging. Keep packaging film away from children.

FILTER CHANGE → p. 3 · no tools · after 6 months or 4,000 l at the latest

- 1 Turn off the water.
- 2 Hold the shower head and unscrew the union nut from the handle tube. The handle can stay on the hose.
- 3 Pull the old filter cartridge upwards out of the handle tube. Residual water may run out.
- 4 Unpack the new cartridge – hold it by the light-coloured housing only and do not touch the membrane side.
- 5 Push the cartridge fully into the handle tube with the membrane side and O-ring facing up (towards the shower head).
- 6 Hand-tighten the shower head, let the water run for approx. 30 seconds and check for leaks. Note the date and dispose of the old cartridge with household waste.

Order replacement filter

Replacement filter for Safety Chrom, item no. 1071 – scan the QR code or order online:

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Scan the QR code

TECHNICAL DATA

Item number	1071 · EAN 4260206241423
Compatible with	Safety Chrom
Filter type	Hollow-fibre membrane filter (sterile filter)
Pore size	0.2 µm
Retention	> log 7 (99.99999 %) to ASTM F838
Flow rate	approx. 6–10 l/min (depending on water pressure)
Service life	4,000 l or max. 6 months
Water temperature	5–60 °C continuous, max. 70 °C for 30 min
Chlorine resistance	up to 10 ppm continuous
Filter cartridge	approx. 3 × 3 × 10 cm
Tests	ASTM F838; integrity test of every cartridge to ASTM D6908-06; KTW-BGWL; ACS; in line with TWIN-12 recommendation

FREQUENTLY ASKED QUESTIONS (FAQ)

- ? How often must the filter cartridge be replaced?**
After 6 months or 4,000 litres at the latest, whichever comes first – even if the water still flows well. Record every change in the log on p. 20.
- ? The water flow is getting weaker. Is that normal?**
Yes. The membrane retains particles and microorganisms and gradually clogs – faster in older pipework. Replace the cartridge. An optional pre-filter can extend its service life.
- ? Does the cartridge fit other Sanquell products?**
No. The replacement filter (item no. 1071) only fits the Safety Chrom – not the Safety Chrom Premium, Safety Tap, Safety Tap neo or Longlife.
- ? What should I consider when the pipes are disinfected?**
Unscrew the shower head before thermal disinfection (continuously above 60 °C or above 70 °C) or chlorination. Afterwards, let the water run at a washbasin for approx. 5 minutes before showering again.
- ? Does the filter also remove limescale?**
No. Dissolved limescale passes through the membrane. Limescale on the silicone nozzles can easily be rubbed off with your fingers.
- ? Can I clean or reuse the cartridge?**
No. The cartridge is a single-use product and must not be opened, backflushed or reused.

CLEANING AND CARE

Clean the outer surfaces with a soft, damp cloth and a little mild soap. Do not use abrasive, solvent-based or acidic cleaners (e.g. descalers). Remove limescale from the silicone nozzles by rubbing them with your fingers.

DISPOSAL

Dispose of used filter cartridges with household waste. Dispose of the shower head at the end of its service life in accordance with local regulations. Recycle the packaging (paper/cardboard).

MANUFACTURER / PRODUCT SAFETY (GPSR)

Sanquell GmbH

Sieker Berg 9 · 22962 Siek · Germany
compliance@sanquell.de · www.sanquell.de

Product identifier: item number see technical data, batch (LOT) see packaging. Product safety enquiries:
compliance@sanquell.de

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