
Safety Tap

Legionellenfilter Wasserhahn & Badewanne · Anti-Legionella filter for tap & bathtub

DE	Gebrauchsanleitung	S. 5
EN	Instructions for use	p. 8
FR	Mode d'emploi	p. 11
IT	Istruzioni per l'uso	p. 14
ES	Instrucciones de uso	pág. 17

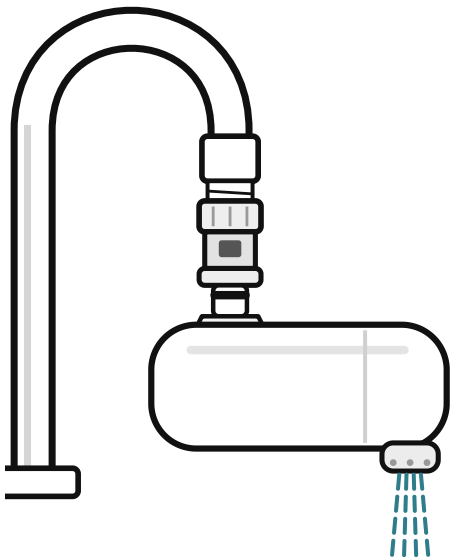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0,2 µm





Porengröße · pore size

log 6










99,9999 %

8 l/min








max. 2 Monate · months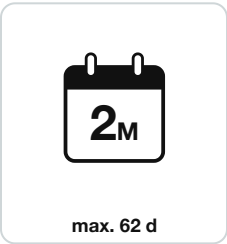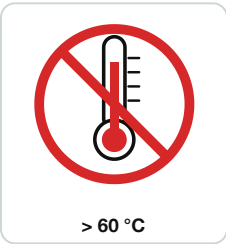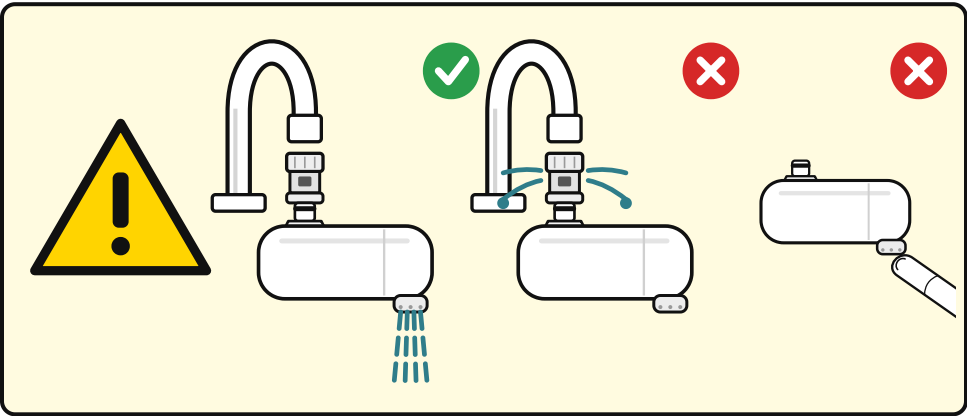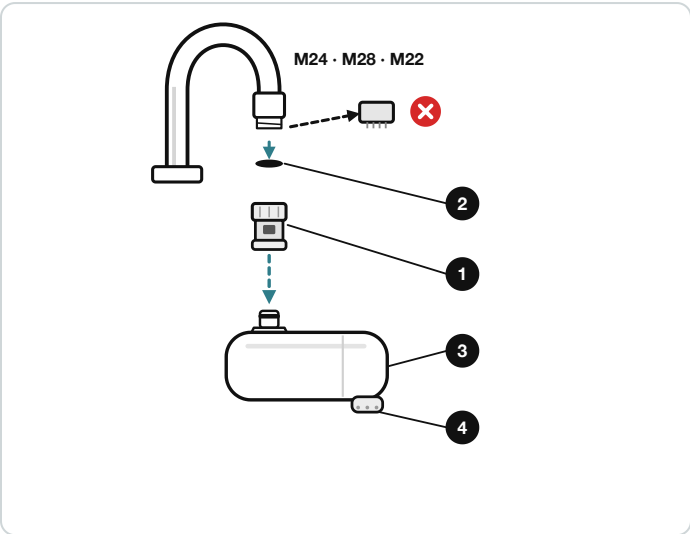

A Installation

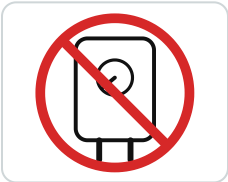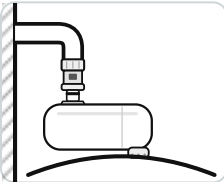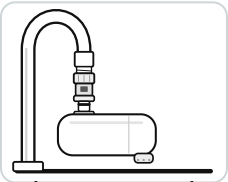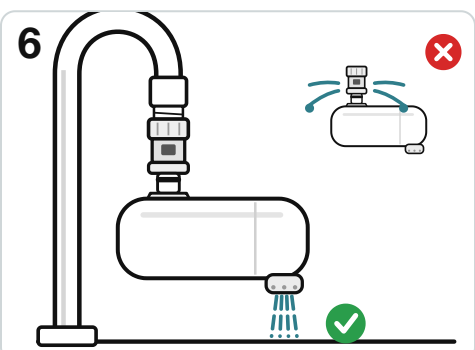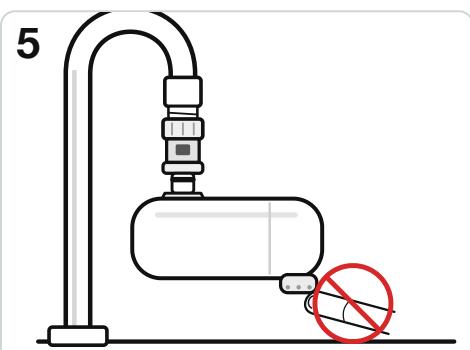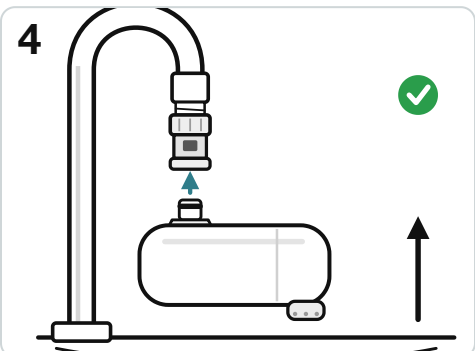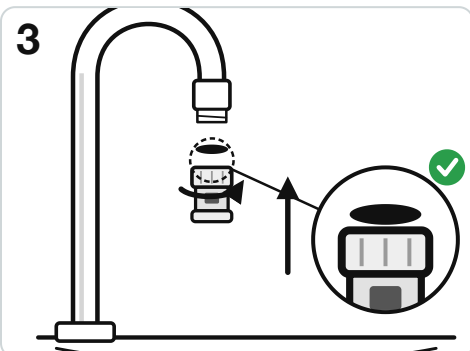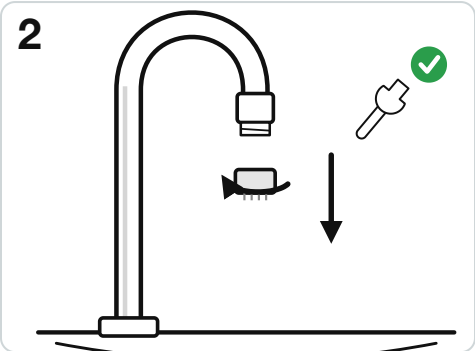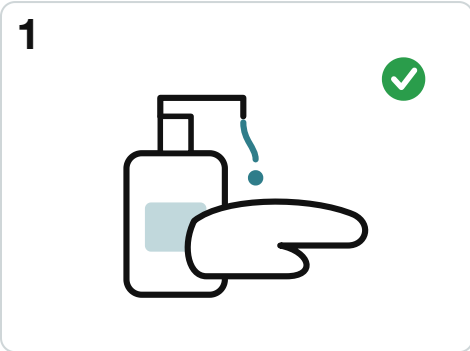B Filter

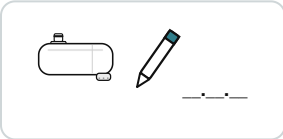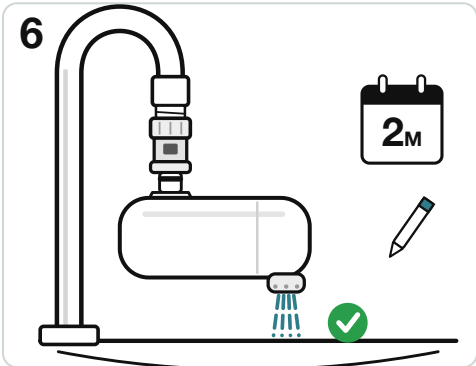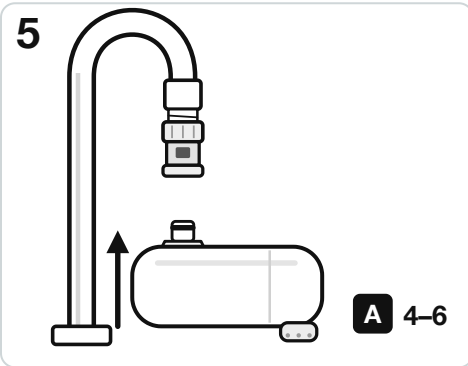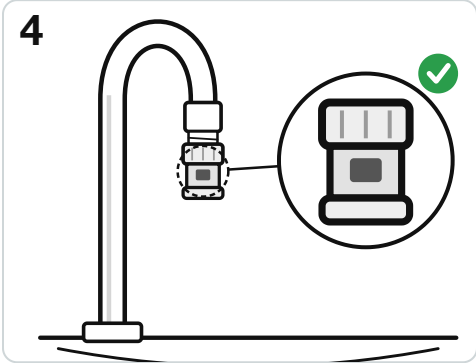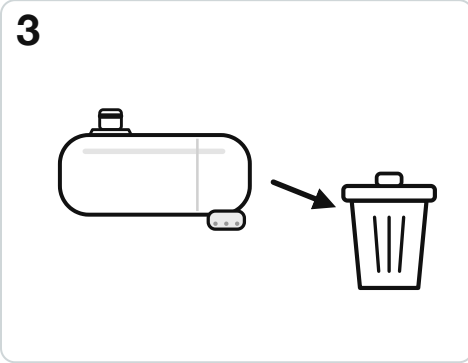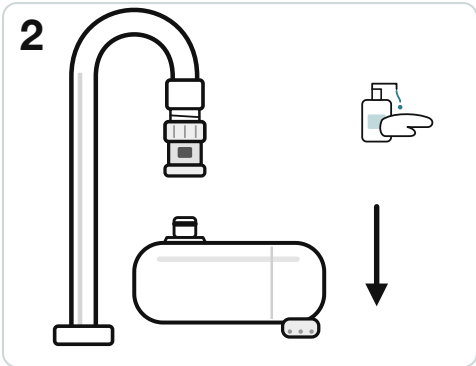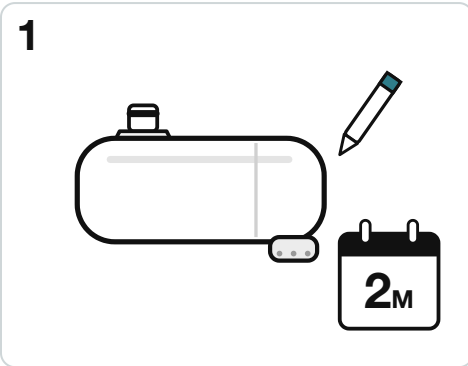4260206249962
 Ersatzfilter · Replacement filter ·
 Filtre de rechange · Filtro di
 ricambio · Filtro de recambio

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INTENDED USE

The Safety Tap is a rod-shaped membrane sterile filter (0.2 µm) for washbasins and bathtubs. It is pushed onto a safety adapter directly at the outlet of the fitting and mechanically retains Legionella, Pseudomonas and E. coli before the water is atomised and can be inhaled. Nothing is released into the water; taste and minerals remain unchanged. The filter only protects the outlet on which it is mounted and does not replace remediation of the building's water system.

SCOPE OF DELIVERY (COMPLETE SET) AND COMPONENTS → p. 2

- 1 Safety adapter (quick-release coupling) 2 Seal 3 Safety Tap filter 4 Filter outlet

SAFETY INSTRUCTIONS

Adapter required: The filter only fits the safety adapter from the complete set. Improper handling can damage the filter and its function.

- **Temperature:** Max. 60 °C. Remove the filter before thermal disinfection of the building's water system.
- **Pressure:** Max. 5 bar. Do not connect to low-pressure systems, e.g. unpressurised under-sink water heaters or instantaneous heaters with an open outlet.
- **Service life:** Replace the filter after 2 months at the latest, earlier depending on water quality. Cleaning or reprocessing the filter is not intended.
- **Hygiene:** Wash or disinfect your hands before installation and filter change. Do not touch the filter outlet with your fingers.
- **Tools:** Only for unscrewing the aerator. Push the filter on by hand.
- **Space required:** The rod filter is 105 mm long. There must be enough space below the outlet – in tight spaces the compact Safety Tap neo is more suitable.

INSTALLATION → p. 3 · push filter on without tools

- 1 Wash or disinfect your hands.
- 2 Unscrew the aerator at the front of the outlet – tools may be used for this.
- 3 Screw the safety adapter with seal onto the outlet thread in place of the aerator.
- 4 Push the filter from below onto the plug connection of the adapter until it sits firmly.
- 5 Do not touch the filter outlet with your fingers.
- 6 Turn on the water and check for flow and leaks. Note the installation date in the log (p. 20).

Places of use: washbasins and bathtubs with enough space below the outlet. Not suitable: flat waterfall outlets without aerator, square outlets, pull-out kitchen sprays, angular bath spouts without thread and recessed CACHÉ aerators.

Adapter: 28 mm male = M28, 23.8 mm male = M24 (complete set 24/28 male); 21.8 mm female = M22 (complete set M22 female); 20, 18.5, 16.5 or 16 mm = multi-adapter set.

FILTER CHANGE → p. 4 · no tools · after 2 months at the latest

- ➊ Change after 2 months at the latest, earlier depending on water quality.
- ➋ Close the tap and wash or disinfect your hands.
- ➌ Pull the old filter downwards off the adapter. The adapter stays on the fitting.
- ➍ Dispose of the old filter with household waste.
- ➎ Check the adapter for cleanliness, seal and firm fit.
- ➏ Push on the new filter as in steps A4–A6, check it and note the date.

**Order replacement filter**

Replacement filter Safety Tap (4260206249962) – scan the QR code or order online:

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**Fit & adapters**

Thread sizes and adapters on the product page

TECHNICAL DATA

Item number (set)	1391 (M24/M28 male) · 145-M22-Set (M22 female) · 145-Multi
Replacement filter	Safety Tap, 4260206249962
Adapter	M24/M28 male, M22 female or multi-adapter set (M16, M16.5, M18.5, M20, M22, M24)
Filter type	Membrane sterile filter (hollow-fibre membrane)
Pore size	0.2 µm
Retention	log 6 (99.9999 %)
Flow rate	up to 8 l/min
Service life	up to 2 months, depending on water quality
Max. operating pressure	5 bar (not for low-pressure systems)
Max. water temperature	60 °C
Dimensions	105 × 44 × 51 mm
Design	Rod filter

FREQUENTLY ASKED QUESTIONS (FAQ)

- ? How often must the filter be replaced?**
After 2 months at the latest, earlier depending on water quality. The adapter stays on the fitting. Record every change in the log on p. 20.
- ? Which adapter do I need?**
Unscrew the aerator and measure the diameter: 28 mm male = M28, 23.8 mm male = M24, 21.8 mm female = M22. For 20, 18.5, 16.5 or 16 mm there is the multi-adapter set. If in doubt, send Sanquell a photo of the outlet.
- ? Is a replacement filter enough to get started?**
No. The complete set is always needed for the first purchase – only it contains the safety adapter without which no filter fits the fitting.
- ? Can I switch to the Safety Tap neo?**
Yes, at any time. Both filters have the same plug connection and fit the same adapter. Simply order the neo Lean replacement filter at the next change.
- ? Which fittings are not suitable?**
Flat waterfall outlets without aerator, square outlets with a rectangular jet, pull-out kitchen sprays, angular bath spouts without thread and recessed CACHÉ aerators with a coin slot.
- ? Will the water flow more slowly?**
Slightly, because the water passes through a membrane with 0.2 µm pores. The Safety Tap delivers up to 8 l/min. A weaker flow towards the end of the service life is normal.
- ? Can I run hot water through it?**
Up to 60 °C is no problem. For hotter water, such as thermal disinfection of the building's water system, remove the filter beforehand.
- ? May I use the filter with an under-sink water heater?**
No. The filter must not be operated on unpressurised under-sink water heaters or instantaneous heaters with an open outlet.
- ? Can I take the filter with me?**
Yes. Filter and adapter can be removed from the outlet and fitted again, e.g. in a holiday flat – provided the fitting there has a suitable thread.
- ? Does the filter replace remediation?**
No. It protects the outlet on which it is mounted. If Legionella is detected, it is a recognised immediate measure until the cause has been eliminated.

MAINTENANCE AND CLEANING

Within its service life, the filter is maintenance-free; cleaning or reprocessing is not intended. Wipe the outer surfaces with a damp cloth or a common surface disinfectant if necessary. Check the adapter and seal at every filter change and replace them if necessary.

DISPOSAL

Dispose of the used filter with household waste. Recycle the packaging.

MANUFACTURER / PRODUCT SAFETY (GPSR)

Sanquell GmbH

Sieker Berg 9 · 22962 Siek · Germany
compliance@sanquell.de · www.sanquell.de

Product identifier: item number see technical data, batch (LOT) see filter or packaging. Product safety enquiries: compliance@sanquell.de

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