
Safety Tap NEO

Legionellenfilter Wasserhahn & Badewanne · Anti-Legionella filter for tap & bathtub

DE	Gebrauchsanleitung	S. 5
EN	Instructions for use	p. 8
FR	Mode d'emploi	p. 11
IT	Istruzioni per l'uso	p. 14
ES	Instrucciones de uso	pág. 17

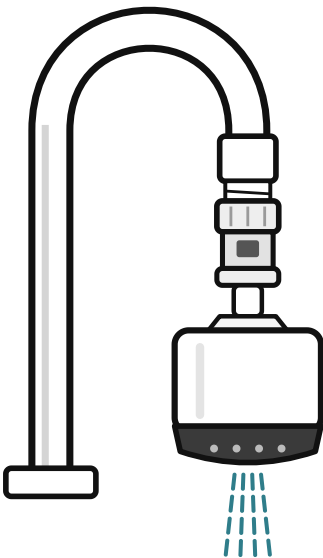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0,2 µm




Porengröße · pore size

7 log

ASTM F838-20

2 M

max. 62 Tage · days

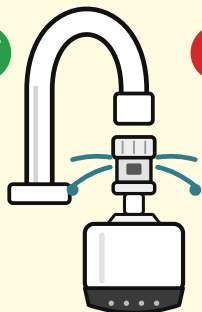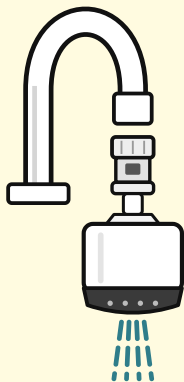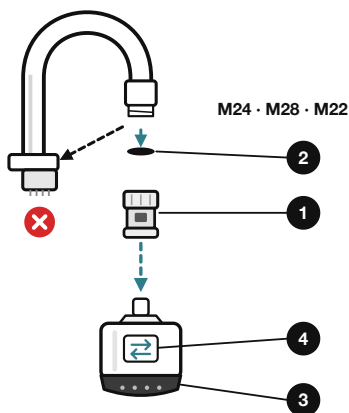> 60 °C








0 bar








max. 62 d

A Installation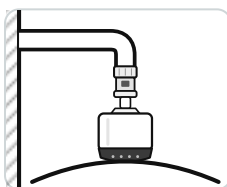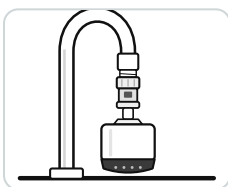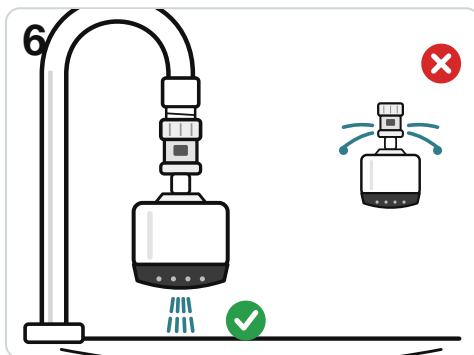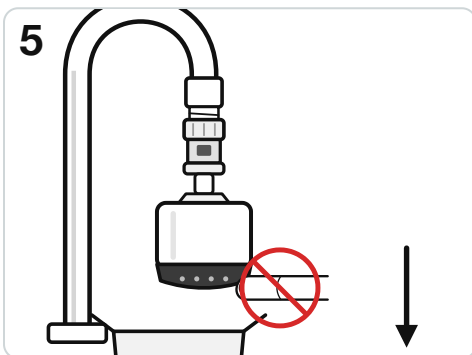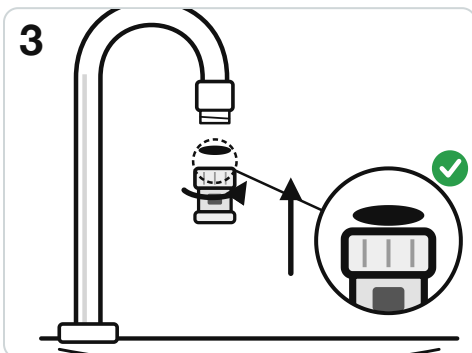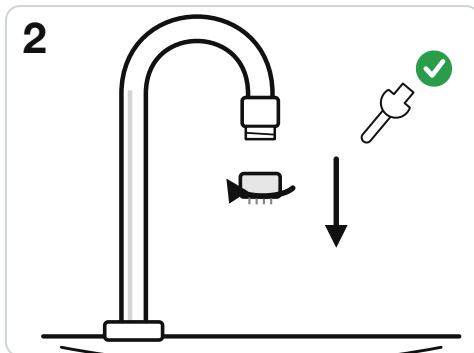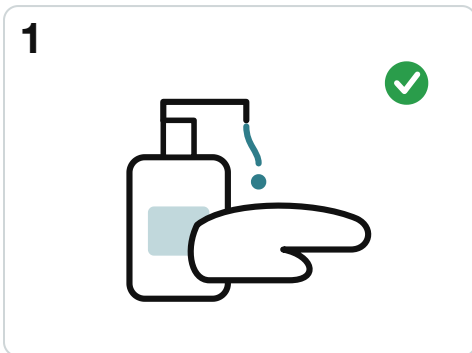

B Filter

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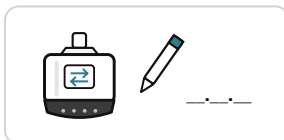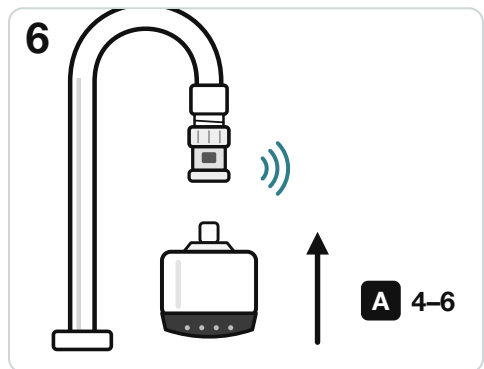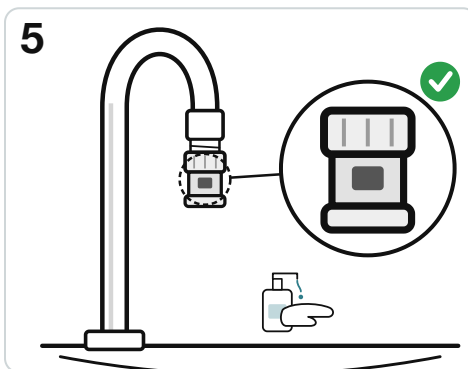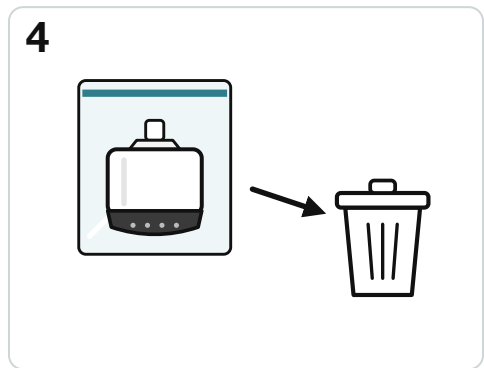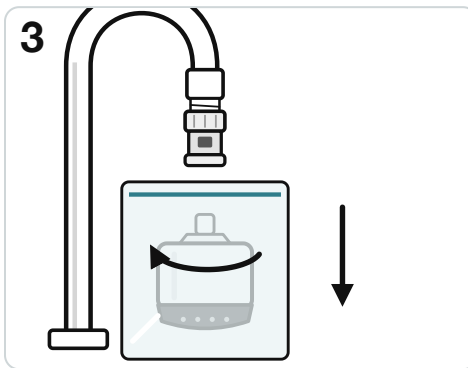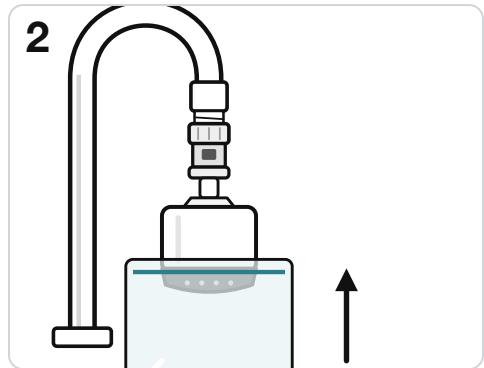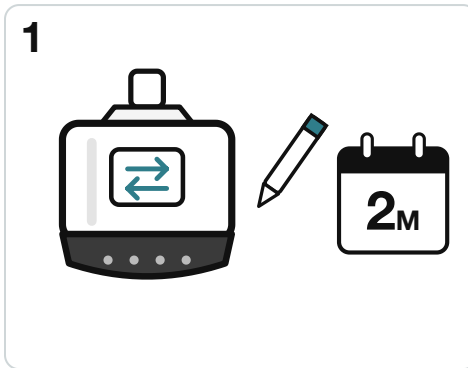INTENDED USE

The Safety Tap NEO is a point-of-use tap sterile filter (flat membrane, 0.2 µm) for use in the event of Legionella contamination. It serves to prevent bacterial infections caused by Legionella pneumophila and also protects against Pseudomonas aeruginosa, Stenotrophomonas maltophilia and Staphylococcus aureus – especially for immunocompromised persons. It works by filtering the water directly at the outlet and by protecting against retrograde contamination. The filter is attached to the tap or bath mixer via a quick-release coupling. It does not replace remediation measures for a contaminated drinking water system.

SCOPE OF DELIVERY AND COMPONENTS → p. 2

- 1 Quick-release coupling (adapter)
- 2 Seal
- 3 Safety Tap NEO filter
- 4 Change label

SAFETY INSTRUCTIONS

Attention: Improper handling can damage the filter and its function. Sanquell accepts no liability for defects caused by improper use.

- **Hygiene:** Disinfect your hands before installation and filter change. Do not touch the filter outlet with your fingers.
- **Temperature:** Max. 60 °C; 75 °C only briefly and for no more than 90 minutes in total over the entire service life.
- **Pressure:** Max. 5 bar. Do not connect the filter to low-pressure systems (e.g. unpressurised small water heaters).
- **Service life:** Replace the filter after 62 days (2 months) at the latest. Depending on the water quality, the service life may be shorter – see filter label.
- **Tools:** Only for unscrewing the aerator. Always insert the filter by hand.
- **Damage:** If the filter deviates from its specification or the packaging is damaged, do not use it; send photos and a description to hallo@sanquell.de.

INSTALLATION → p. 3 · insert filter without tools

- 1 Disinfect your hands.
- 2 Unscrew the aerator from the tap – tools may be used for this.
- 3 Screw the quick-release coupling with seal onto the outlet thread.
- 4 Open the filter packaging. Insert the filter into the coupling with a slight twisting motion until it audibly clicks into place.
- 5 Remove the packaging without touching the filter outlet.
- 6 Turn on the water and check the installation for flow and leaks. Note the change date on the change label and in the log (p. 20).

Places of use: washbasin, kitchen sink and bathtub – on fittings whose aerator can be unscrewed.

Adapter: The thread at the outlet determines the coupling: male thread M24 or M28, female thread M22. Further sizes in the multi-adapter set or on request.

FILTER CHANGE → p. 4 · no tools · after 2 months at the latest

- 1 Keep an eye on the change date: the service life is max. 2 months (62 days).
- 2 Close the tap and pull the pressure-seal bag over the filter.
- 3 Pull the filter out of the coupling with a slight twisting motion; it stays inside the bag.
- 4 Seal the bag and dispose of the filter with household waste.
- 5 Disinfect your hands. Check the coupling and connections for function and proper condition.
- 6 Insert the new filter as in steps A4–A6, check it and note the new change date.

**Order replacement filter**

Replacement filter neo Lean, item no. SQ-1146 – scan the QR code or order online:

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**Adapters & accessories**

Product page with all adapter sizes

TECHNICAL DATA

Item number (set)	SQ-1145 (M24/M28 male) · SQ-1145-M22 (M22 female) · SQ-1145-Multi
Replacement filter	neo Lean, item no. SQ-1146
Couplings	M24 male with AquaStop (100298) · M22 female with AquaStop (100296) · M24 male short, without AquaStop (102668)
Filter type	Sterile filter, flat membrane
Pore size	0.2 µm
Retention	7 log steps ($\geq 10^7$ CFU B. diminuta/cm ² , ASTM F838-20)
Flow rate	15.5 l/min at 5 bar · 13.1 l/min at 3 bar
Service life	up to 62 days (2 months), depending on water quality
Max. operating pressure	5 bar (not for low-pressure systems)
Max. operating temperature	60 °C (75 °C ≤ 90 min over the service life)
Dimensions	49.3 mm × Ø 52 mm
Connection	Quick-release coupling

FREQUENTLY ASKED QUESTIONS (FAQ)

- ? How often must the filter be replaced?**
After 62 days (2 months) at the latest; earlier depending on water quality. Note the date on the change label and in the log on p. 20.
- ? Which coupling fits my tap?**
Unscrew the aerator and measure the thread: male thread approx. 24 mm = M24, approx. 28 mm = M28, female thread approx. 22 mm = M22. Further sizes are available in the multi-adapter set.
- ? Does the coupling have to be removed when changing the filter?**
No. The coupling stays on the tap; only the filter is replaced.
- ? Can I use the filter on any fitting?**
On fittings with a removable aerator and a water pressure of up to 5 bar. Do not connect it to low-pressure systems such as unpressurised small water heaters.
- ? The flow rate is decreasing. What does that mean?**
The membrane retains particles and germs and gradually clogs. If the flow drops noticeably, replace the filter before the end of its service life.
- ? Water is leaking at the coupling. What should I do?**
Close the tap, check the seal, re-tighten the coupling and re-insert the filter until it audibly clicks into place. If the connection still leaks, replace the coupling.
- ? May warm water flow through the filter?**
Yes, up to 60 °C. 75 °C is only permitted briefly and for no more than 90 minutes in total over the service life.
- ? How do I clean the filter?**
Wipe the outside with a common surface disinfectant – ideally daily. Do not touch the filter outlet with your fingers.
- ? Does the filter change the water?**
No chemicals are added, and the minerals in the water are retained.
- ? Can I reuse a used filter?**
No. Dispose of used filters in the pressure-seal bag with household waste.

MAINTENANCE AND CLEANING

Within its service life, the filter is maintenance-free. It can be treated externally with common surface disinfectants; daily wipe disinfection is recommended to prevent retrograde contamination. Small amounts of unfiltered water enter the coupling during each filter change and may cause deposits – check adapters and connections regularly and replace them if necessary.

DISPOSAL

Dispose of the used filter in the sealed pressure-seal bag with household waste. Recycle the packaging.

MANUFACTURER / PRODUCT SAFETY (GPSR)

Sanquell GmbH

Sieker Berg 9 · 22962 Siek · Germany
compliance@sanquell.de · www.sanquell.de

Product identifier: item number see technical data, batch (LOT) see filter label or packaging. Product safety enquiries: compliance@sanquell.de

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