Longlife Wechselfilter

Longlife

DE	Gebrauchsanleitung	S. 5
EN	Instructions for use	p. 8
FR	Mode d'emploi	p. 11
IT	Istruzioni per l'uso	p. 14
ES	Instrucciones de uso	pág. 17

1552

0,1 µm


Porenweite · pore size

log 7

99,99999 %

6 M



max. 6 000 l

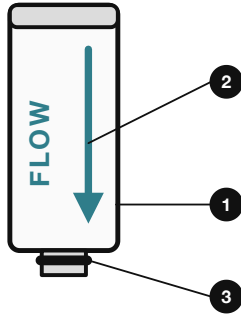15 s

1-5 bar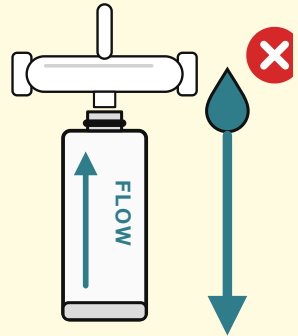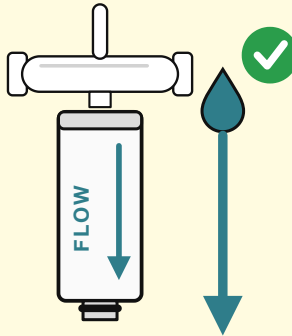

> 60 °C








< 5 °C

max. 6 000 l

B Filter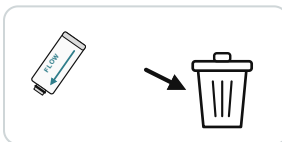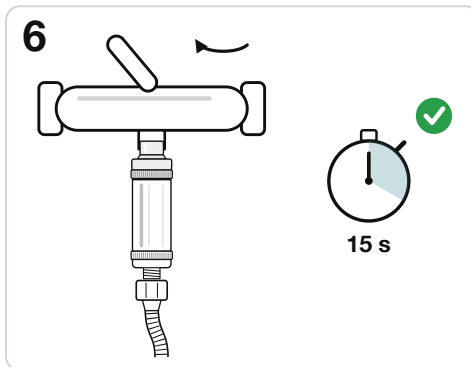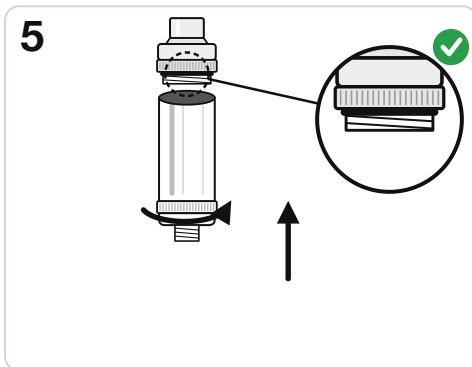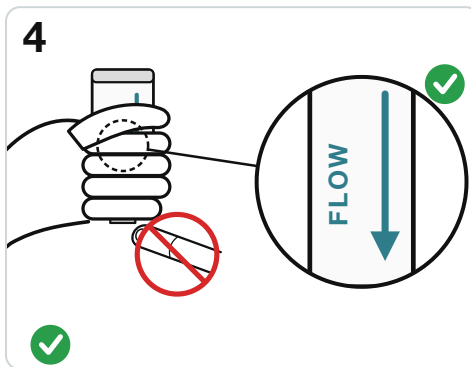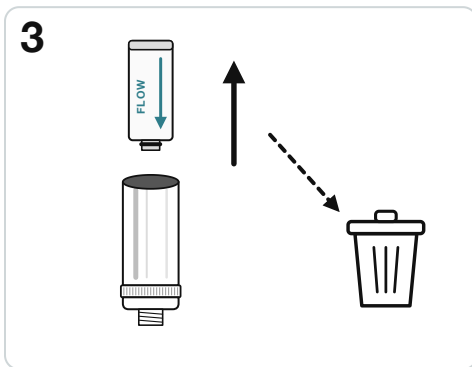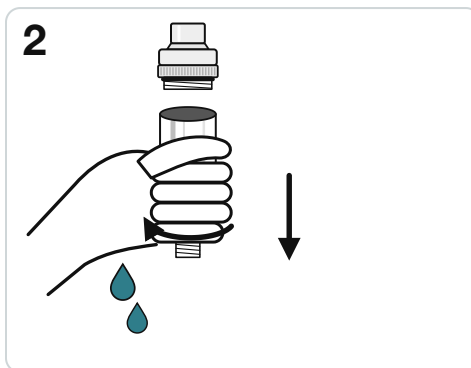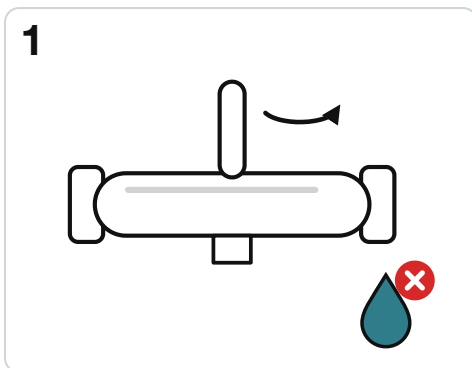


1552
 Wechselfilter Longlife ·
 Replacement filter · Filtre de
 rechange · Filtro di ricambio ·
 Filtro de recambio


Longlife


Longlife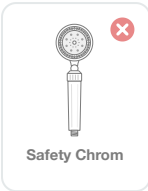


Safety Chrom

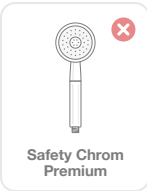
Safety Chrom Premium



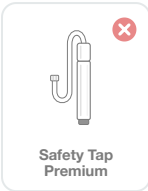
Safety Tap Premium

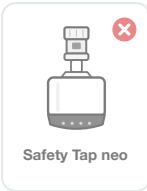Safety Tap neo

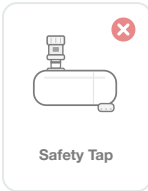Safety Tap

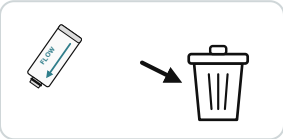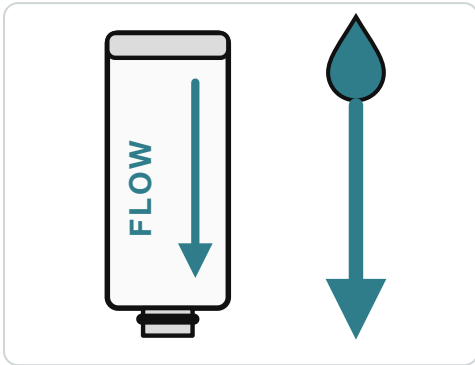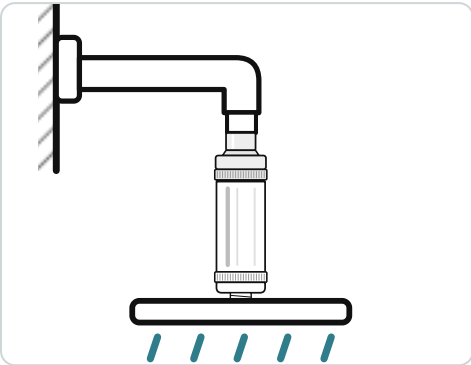1552
Wechselfilter Longlife ·
Replacement filter · Filtre de
rechange · Filtro di ricambio ·
Filtro de recambio

1552

Wechselfilter Longlife · Replacement filter · Filtre de
rechange · Filtro di ricambio · Filtro de recambio

► **Video**

Filterwechsel · Filter change · Remplacement ·
Sostituzione · Cambio

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INTENDED USE

The Longlife replacement filter is a sterile filter cartridge (hollow-fibre membrane, 0.1 µm, 0.24 m² filter area) for the chrome-plated housing of the Longlife anti-Legionella filter. It retains Legionella, Pseudomonas and other bacteria. Only the cartridge is replaced – housing, shower head and rain shower stay in place; the “FLOW” arrow shows the installation direction.

- ❶ Filter cartridge
- ❷ “FLOW” arrow – in direction of flow
- ❸ Sealing ring

Fits: Longlife

Does not fit: Safety Chrom, Safety Chrom Premium, Safety Tap Premium, Safety Tap neo, Safety Tap

SAFETY INSTRUCTIONS

Flow direction: The “FLOW” arrow on the cartridge must point in the direction of flow – from the mixer to the hose or shower head. Never insert the cartridge against the direction of flow.

- **Temperature and pressure:** Up to 60 °C continuously, up to 70 °C briefly (max. 30 minutes); water pressure 1–5 bar. Unscrew the filter before thermal disinfection.
- **Hygiene:** Only hold a new cartridge by its body; do not touch the connections or openings.
- **Replacement interval:** Replace the filter cartridge after 6 months or 6,000 litres at the latest, whichever comes first.
- **Installation:** Hand-tighten only. Do not use tools or sealing tape.
- **Do not tamper:** Do not open, backflush or reuse the cartridge. Do not use damaged or leaking products.
- **Storage:** Store frost-free, dry and in the original packaging. Keep packaging film away from children.

FILTER CHANGE → p. 3 · no tools · after 6 months or 6,000 l at the latest

- ❶ Turn off the water.
- ❷ Unscrew the filter housing by hand. Residual water may run out.
- ❸ Remove the old filter cartridge.
- ❹ Hold the new cartridge by its body only and insert it so that the “FLOW” arrow points in the direction of flow.
- ❺ Check that the seals are clean and correctly seated, then hand-tighten the housing.
- ❻ Let the water run for 15 seconds and check the connections for leaks. Note the date and dispose of the old cartridge with household waste.

Rain shower: The flow direction may be reversed here. Open the housing by hand and turn the cartridge if necessary so that the “FLOW” arrow points in the direction of flow.

Order replacement filter

Longlife replacement filter, item no. 1552 – scan the QR code or order online:

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Scan the QR code

TECHNICAL DATA

Item number	1552 - EAN 4260206249795
Compatible with	Longlife
Filter type	Hollow-fibre membrane filter (sterile filter)
Pore size	0.1 µm
Retention	log 7 (99.99999 %)
Flow rate	up to 10 l/min
Service life	6,000 l or max. 6 months
Water temperature	5–60 °C continuous, max. 70 °C for 30 min
Water pressure	1–5 bar
Chlorine resistance	up to 300 ppm for 12 h
Membrane area	0.24 m²
Dimensions	10.7 × 4.5 × 7.1 cm
Weight	72 g
Test	ASTM F838-15a with Legionella pneumophila, Eurofins Biolab

FREQUENTLY ASKED QUESTIONS (FAQ)

- ? How often must the filter cartridge be replaced?**
After 6 months or 6,000 litres at the latest, whichever comes first. Record every change in the log on p. 20.
- ? Which way round is the cartridge inserted?**
The "FLOW" arrow points in the direction of flow, i.e. from the mixer to the hose or shower head. With rain showers, the cartridge may therefore have to be inserted the other way round.
- ? The water flow is getting weaker. Is that normal?**
Yes. The membrane retains particles and microorganisms and gradually clogs. Replace the cartridge. If old pipes carry rust or sand, an upstream sediment filter protects the cartridge.
- ? The housing is hard to open.**
Limescale can seize the thread. Therefore unscrew and re-tighten the housing now and then. Do not use tools that could damage the surface.
- ? Does the replacement cartridge fit other Sanquell filters?**
No. The Longlife replacement filter (item no. 1552) only fits the Longlife housing, not the Safety shower heads.
- ? Can I clean or reuse the cartridge?**
No. The cartridge is a single-use product and must not be opened, backflushed or reused.

CLEANING AND CARE

Clean the housing with a soft, damp cloth and a little mild soap. Do not use abrasive, solvent-based or acidic cleaners. Tip: unscrew and re-tighten the housing now and then so that limescale does not seize the thread.

DISPOSAL

Dispose of used filter cartridges with household waste. Dispose of the housing at the end of its service life in accordance with local regulations. Recycle the packaging (paper/cardboard).

MANUFACTURER / PRODUCT SAFETY (GPSR)

Sanquell GmbH

Sieker Berg 9 · 22962 Siek · Germany
compliance@sanquell.de · www.sanquell.de

Product identifier: item number see technical data, batch (LOT) see packaging. Product safety enquiries: compliance@sanquell.de

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