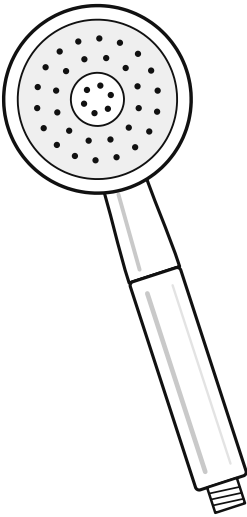
Safety Chrom Premium

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INTENDED USE

The Safety Chrom Premium is a hand shower with an integrated sterile filter (hollow-fibre membrane, 0.085 µm). The filter retains Legionella and other bacteria from the shower water and thus protects against inhaling contaminated aerosols. For use with drinking water only and for connection to standard shower hoses (G ½"). The filter does not replace remediation measures for a contaminated drinking water system.

COMPONENTS → p. 2

- ❶ Shower head with housing
- ❷ Filter handle (replacement filter)
- ❸ O-ring
- ❹ Connection thread G ½" for the shower hose

SAFETY INSTRUCTIONS

Do not drop! Impacts and falls can cause fine, invisible cracks in the housing – especially at the neck between shower head and handle – or in the filter membrane. Filter performance and tightness can then no longer be guaranteed. Do not continue to use the product after a fall.

- **Temperature:** For water at 5–70 °C only. Unscrew the shower head before any thermal disinfection of the pipes (> 70 °C).
- **Hygiene:** Do not touch the open filter ends or the thread of the filter handle. Only hold a new filter handle by its chrome housing.
- **Replacement interval:** Replace the filter handle after 6 months or 6,500 litres at the latest, whichever comes first. Replace it earlier if the flow rate drops noticeably.
- **Installation:** Hand-tighten only. Do not use tools or sealing tape.
- **Do not tamper:** Do not open or backflush the filter handle and do not reuse it in other devices. Do not use damaged or leaking products.
- **Storage:** Store frost-free, dry and in the original packaging. Keep packaging film away from children.

INSTALLATION → p. 3 · approx. 2 minutes · no tools · no sealing tape

- ❶ Turn off the water.
- ❷ Unscrew the existing shower head from the shower hose.
- ❸ Check that the washer in the hose nut is present and undamaged.
- ❹ Hand-tighten the shower hose onto the connection thread of the filter handle.
- ❺ Turn on the water, let it run for approx. 30 seconds and check the connection for leaks.
- ❻ Note the installation date in the filter change log (p. 20).

FILTER CHANGE → p. 4 · approx. 25 seconds · after 6 months or 6,500 l at the latest

- 1 Turn off the water.
- 2 Unscrew the shower hose from the filter handle.
- 3 Hold the shower head and unscrew the old filter handle from the housing.
- 4 Unpack the new filter handle – hold it by the chrome housing only. Check that the O-ring is seated correctly.
- 5 Hand-tighten the new filter handle into the housing.
- 6 Reconnect the hose, let the water run for approx. 30 seconds and check for leaks. Note the date and dispose of the old filter handle.

**Order replacement filter**

Filter handle Safety Premium, item no. SQ-442 – scan the QR code or order online:

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Scan the QR code

TECHNICAL DATA

Item number	SQ-444-15-W (Ø 15 cm) · SQ-444-7-W (Ø 7 cm)
Replacement filter	Filter handle Safety Premium, SQ-442
Filter type	Hollow-fibre membrane filter
Pore size	0.085 µm
Retention	log 8 (99.999999 %)
Flow rate	up to 10 l/min
Service life	6,500 l or max. 6 months
Water temperature	5–70 °C
Chlorine resistance	up to 300 ppm for 12 h
Connection	G ½"
Dimensions	35 × 15 cm (Ø 15 cm) · 27 × 7 cm (Ø 7 cm)
Handle	Ø 3.5 × 17.5 cm
Weight	0.3 kg
Spray modes	3 (normal, rain, jet)
Tested to	ASTM F838-05, ASTM F838-15ae1, NSF P376, ASSE LEC 2011-2022; in line with TWIN-12 recommendation

FREQUENTLY ASKED QUESTIONS (FAQ)

- ? How often must the filter handle be replaced?**
After 6 months or 6,500 litres at the latest, whichever comes first. Record every change in the log on p. 20.
- ? The water flow is getting weaker. Is that normal?**
Yes. The membrane retains particles and microorganisms and gradually clogs – faster in older pipework. Replace the filter handle, even before the 6 months are up.
- ? Does the shower head fit my shower hose?**
Yes, it fits all standard shower hoses with a ½-inch thread (G ½"). No tools or sealing tape are required.
- ? Why is the whole handle replaced?**
The sterile filter is permanently built into the handle, so you never come into contact with the membrane when changing it. The shower head is reused.
- ? Which filter handle fits?**
The Safety Premium filter handle (item no. SQ-442) fits the Safety Chrom Premium in 7 cm and 15 cm as well as the Safety Tap Premium – not the Safety Chrom, Safety Tap, Safety Tap neo or Longlife.
- ? Water is dripping at the connection. What can I do?**
Turn off the water, check the washer in the hose nut and the O-ring of the filter handle for correct fit and damage, and hand-tighten the connection.
- ? What should I consider when the pipes are disinfected?**
Unscrew the shower head before thermal disinfection (above 70 °C). The filter withstands chlorine up to 300 ppm for 12 hours; for stronger chlorination, also remove the shower head beforehand.
- ? Does the filter also remove limescale?**
No. Dissolved limescale passes through the membrane. Limescale on the spray nozzles can be rubbed off with your fingers.
- ? The shower head was dropped. May I continue to use it?**
No. Cracks may have formed even if no damage is visible. Do not continue to use the product (see safety instructions).
- ? Can I reuse a used filter handle?**
No. A used filter handle must not be opened, backflushed or reused in another device.

CLEANING AND CARE

Clean the outer surfaces with a soft, damp cloth and a little mild soap. Do not use abrasive, solvent-based or acidic cleaners (e.g. descalers). Remove limescale from the spray nozzles by rubbing them with your fingers.

DISPOSAL

Dispose of the used filter handle and shower head with residual household waste in accordance with local regulations. Recycle the packaging (paper/cardboard).

MANUFACTURER / PRODUCT SAFETY (GPSR)

Sanquell GmbH

Sieker Berg 9 · 22962 Siek · Germany
compliance@sanquell.de · www.sanquell.de

Product identifier: item number see technical data, batch (LOT) see packaging. Product safety enquiries: compliance@sanquell.de

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