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Adapter and functional materials

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Quick-release coupling M24, external thread (incl. seal), with AquaStop, (Art. No. 100298)

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Quick-release coupling M22, internal thread (incl. seal), with AquaStop, (Art. Nr. 100296)

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Further adapters and functional materials on request

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The Safety Tap NEO is a terminal tap sterile filter⁽¹⁾ for use in the event of Legionella infection. It is used to prevent bacterial infections caused by Legionella pneumoniae. It also protects against Pseudomonas aeruginosa, Stenotrophomonas maltophilia and Staphylococcus aureus in immunocompromised and immunocompromised patients. The effect is achieved by interrupting the transmission path of these germs by filtering the water and protecting against retrograde contamination - through hygienic design and antimicrobial protective functions when using the water taps.

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
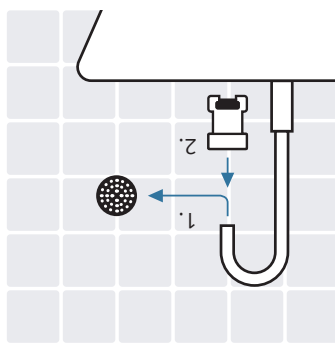
Installation quick release coupling

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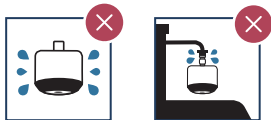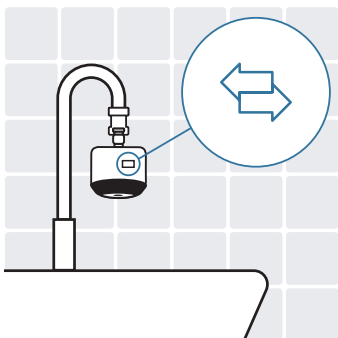
Note next change date on change label. Filter life see filter label/packaging.

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After the service life has expired (2 month), uninstall the filter.

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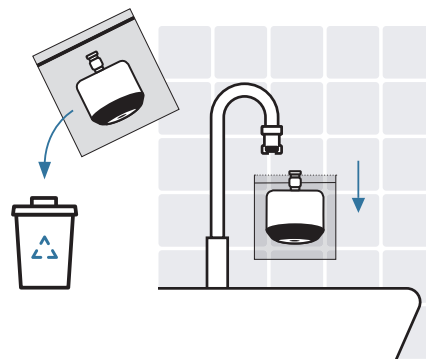
Put the pressure seal bag over the filter and uninstall the filter. Dispose of filter in general waste.

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Within the specified service life, the Safety Tap NEO is maintenance-free. the sterile filter can be treated externally with common surface disinfectants. to prevent retrograde contamination, daily wipe disinfection of the Safety Tap NEO is recommended.

When changing the Safety Tap NEO filter, small amounts of unfiltered water always accumulate. these lead to deposits over time. for this reason, adapters and connections should be checked at regular intervals for functionality and proper condition and replaced as required.

If there is a deviation from the specification of the filter or the packaging of the Safety Tap NEO is damaged, please send photos and a description of the deviations to hallo@sanquell.de

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Flow rate 15.5 l/min at 5 bar, 13.1 l/min at 3 bar

Max. operating pressure 5 bar⁴

Max. operating temp. 60 °C (at 75 °C ≤ 90 min over operating time)

Connection Quick-release coupling

² Gemessen nach internationalem Standard ASTM F838-20 – Standard Test Method for Determining Bacterial Retention of Membrane Filters Utilized for Liquid Filtration.

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² Measured according to international standard ASTM F838-20 - Standard Test Method for Determining Bacterial Retention of Membrane Filters Utilised for Liquid Filtration.

³ Depending on the water quality

⁴ The Safety Tap NEO must not be connected to low-pressure systems.